

PRODRIVE SYSTEM BUR INSERTION AND REMOVAL PROCEDURES



THREE STEPS TO INSERTING THE PRODRIVE BUR

- 01.** Without depressing the push button, gently slide the bur into the spindle until it stops (meets resistance, cannot insert any further).
- 02.** Depress the push button and at the same time, with thumb and forefinger push and rotate the bur into the drive mechanism. Release pressure on the push button. Push/pull the bur to ensure it is seated fully. The bur is now seated in the normal position, and is locked in place.
- 03.** To extend the bur depress the push button pull the bur out about 3 mm until you see the index mark. Release pressure on the push button and push / pull on bur until it clicks into place. The bur is now locked in the extended position.

REMOVAL OF THE PRODRIVE BUR

- 01.** To release the bur from the drive mechanism, depress the push button and at the same time, grasp the bur and remove it. Release the push button.

These photos illustrate the above descriptions. They also show the grooves and index reference markings.

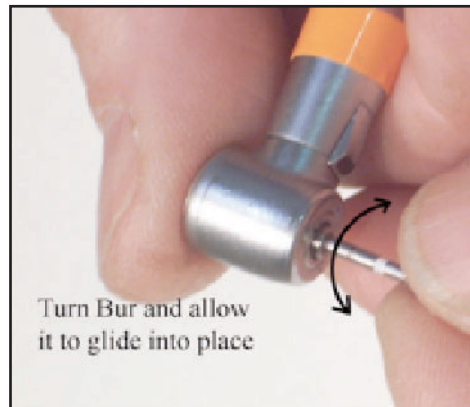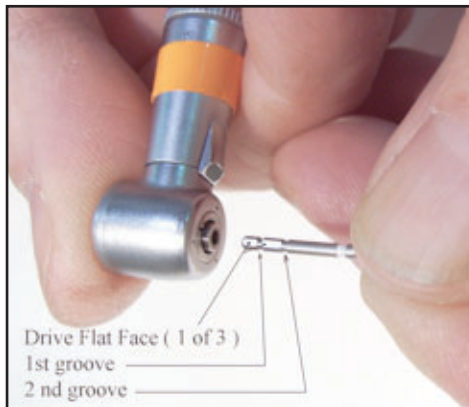

Bur insertion should be done easily and effortlessly. Repeat this procedure until, you are comfortable, and it can be accomplished in a smooth fluid motion

